




**Medtronic**

# LIFEPAK<sup>®</sup> CR<sup>Plus</sup>

Defibrillator



SUDDEN CARDIAC ARREST CAN STRIKE ANYONE, ANYWHERE, ANYTIME.

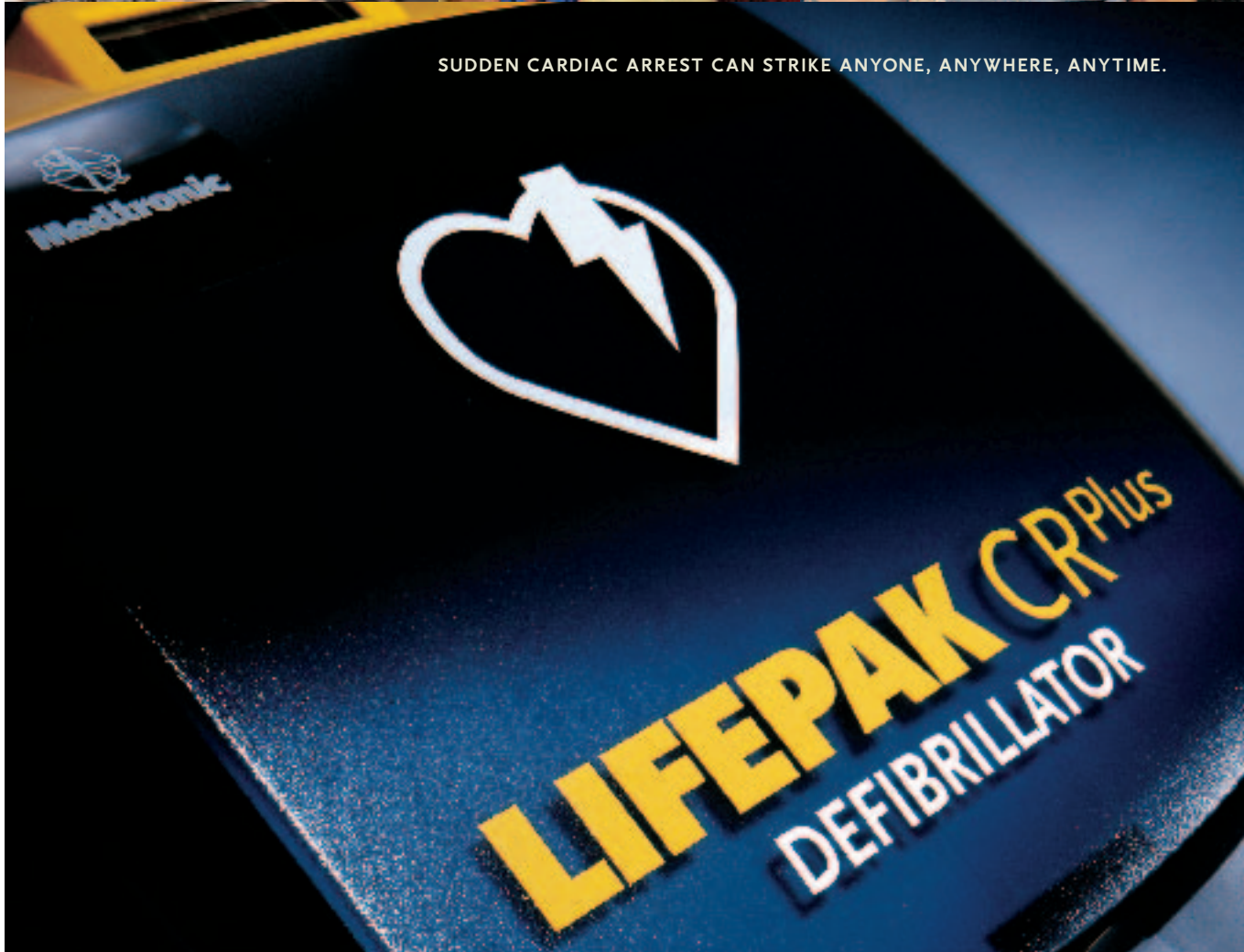

*Lifesaving Made Simple*

# LIFEPAK CR Plus

DEFIBRILLATOR

As many as 450,000 adults and 7,000 children suddenly lose their lives to sudden cardiac arrest each year.<sup>1,2</sup>



**Medtronic created the LIFEPAK CR Plus AED especially for the first person at the scene of a cardiac arrest, when there is the best chance to restart a heart.**

#### THE SMARTEST CHOICE, FOR PEACE OF MIND

▼  
With the LIFEPAK CR Plus AED, you'll have peace of mind knowing you have Medtronic, the world leader in defibrillation, behind you. The LIFEPAK reputation for quality and reliability has earned the trust of the experts in emergency medical care. More hospitals and nine out of the top 10 emergency medical services (EMS) in the U.S. rely on LIFEPAK defibrillators every day to save lives.

#### THE EASIEST AED TO USE, FOR FASTER TIME TO FIRST SHOCK

▼  
For decades our goal has been to get a lifesaving defibrillation shock to people who need it—sooner—because chances of surviving sudden cardiac arrest are best within the first few minutes of arrest. An AED used quickly and effectively by the first person at the scene can have lifesaving results. Ease of use is vital to delivering the first shock faster.

The LIFEPAK CR Plus AED is the easiest-to-use AED ever developed. Although rescuers should be trained in CPR and AED use, a recent study found that untrained people could deliver a shock faster with the *CR Plus* than with the other AEDs tested.<sup>7</sup> And an independent expert review of five AEDs on the market rated the *CR Plus* No. 1 in ease of use and intuitiveness.<sup>8</sup>

The only fully automatic AED available, the *CR Plus* does everything for you. All you have to do is follow the prompts, apply the preconnected electrode pads and stand back. There is no shock button to push, and the *CR Plus* is designed to deliver a shock only if it determines the heart needs it. For those who want more control, the semi-automatic *CR Plus* comes with a shock button.

#### THE BEST TECHNOLOGY AVAILABLE FOR SAVING MORE LIVES

▼  
Packaged in this easy-to-use device, the LIFEPAK CR Plus AED houses the same advanced therapy that more medical professionals trust and use every day. ADAPTIV™ biphasic technology adjusts therapy to each person's unique needs and provides repeat, escalating shocks (200–360 joules) if the heart doesn't respond to previous shocks.

Infant/Child Reduced Energy Defibrillation Electrodes now make it possible for even the youngest victims of sudden cardiac arrest to receive potentially lifesaving defibrillation.

The *CR Plus* is built to the tough specifications of emergency medical services teams. It can withstand rugged treatment and can travel without injury.

THE SIMPLE-TO-USE LIFEPAK CR PLUS AED HOUSES THE MOST ADVANCED TECHNOLOGY AVAILABLE FOR TREATING SUDDEN CARDIAC ARREST—FOR ANYBODY WHO NEEDS IT.



Survival rates drop 7–10 percent every minute without defibrillation.<sup>3</sup>

The American Heart Association recommends defibrillation within 3–5 minutes or sooner for cardiac arrests occurring outside the hospital.<sup>4</sup>

On average, it takes emergency medical services teams 6–12 minutes to arrive.<sup>5</sup>

A national trial revealed the number of survivors from sudden cardiac arrest in public places approximately doubled when trained laypeople used AEDs.<sup>6</sup>



### THE EASIEST AED TO OWN

▼  
The LIFEPAK CR Plus AED has a battery system that uses an easy-to-change CHARGE-PAK™ battery charger. You can rest assured because a built-in system ensures the *CR Plus* has full power in the event of a sudden cardiac arrest. Synchronized replacement dates for the battery charger and electrodes cut required maintenance efforts in half.

Built for reliability, with careful engineering up-front, a regular self-test offers additional assurance that your *CR Plus* is rescue ready. The easily visible readiness display tells you at a glance if your device needs service.



EVERY DAY, SOMEONE LIKE YOU USES  
AN AED TO HELP SAVE A LIFE.  
ARE YOU PREPARED?

The LIFEPAK CR Plus AED shows and tells you what to do one step at a time, using a “layered approach” that makes each step obvious so you can use the AED quickly and effectively.



## ABOUT MEDTRONIC

Founded in 1949, Medtronic is now the world's leading medical technology company. Medtronic provides lifelong solutions for people living with chronic disease. It also offers a wide variety of products and therapies to help physicians solve the most challenging and limiting medical problems, restoring health, extending life and alleviating pain.

## YOUR COMPLETE AED SOLUTION

Starting an AED program is simple, thanks to LIFEPAK Heart Safe Solutions. This turnkey program provides you with everything you need: the LIFEPAK CR Plus AED, medical authorization, training, completion of paperwork, supply replenishment, on-site professional counseling after an event, and help complying with local, state and federal legislation.

The Heart Safe Solutions program can be tailored specifically to your needs. You may purchase it outright or on a monthly payment plan. It's that simple.

LIFEPAK AEDs are prescription devices. AED users should be trained in CPR and use of the AED. Although not everyone can be saved from sudden cardiac arrest, studies show that early defibrillation can dramatically improve survival rates. The LIFEPAK CR Plus AED may be used on children up to eight years old or 25 kg (55 lbs) only with the Infant/Child Reduced Energy Defibrillation Electrodes.

### References

- Zheng, Z., Croft, J., Giles, W., Mensah, G. 2001. Sudden cardiac death in the United States, 1989-1998. *Circulation*, 104:2158-63.
- Berger, S., Dhalal, A., Freidberg, D.Z. 1999. Sudden cardiac death in infants, children and adolescents. *Pediatric Clinics of North America*, (46) 2: 221-34.
- Cummins, R.O. 1989. From concept to standard-of-care? Review of the clinical experience with automated external defibrillators. *Annals of Emergency Medicine* 18: 1269-75.
- Guidelines 2000 for Cardiovascular Resuscitation and Emergency Cardiovascular Care. 2000. *Circulation* 102 (8) 11-384.
- Medtronic review of published literature.
- Ornato, J.P. and The Public Access Trial Investigators. 2003. The Public Access Defibrillation (PAD) Trial (Abstract) *Circulation*, 108:2723.
- Eames, P., Larsen, P.D., and Galletly, D.C. 2003. Comparison of ease of use of three automated external defibrillators by untrained lay people. *Resuscitation*, 58: 25-30.
- Human Factors International. Expert review of five commercially available AEDs for intuitiveness and ease of use. May 2003.



**Medtronic Physio-Control**  
11811 Willows Road NE  
P. O. Box 97006  
Redmond, WA 98073-9706 USA  
Tel: 425.867.4000  
Fax: 425.867.4121  
www.physiocontrol.com  
www.medtronic.com

**Europe**  
Tolochenaz, Switzerland  
Tel: 41.21.802.7000  
Fax: 41.21.802.7900

**Switzerland**  
Tolochenaz, Switzerland  
Tel: 41.21.803.8000  
Fax: 41.21.803.8099

**Canada**  
Mississauga, Ontario  
Tel: 905.826.6020  
Fax: 905.826.6620

**United Kingdom, Ireland**  
Watford, Great Britain  
Tel: 44.1923.212.213  
Fax: 44.1923.241.004

**France**  
Boulogne-Billancourt, France  
Tel: 33.1.55.38.1700  
Fax: 33.1.55.38.1800

**Germany**  
Dusseldorf, Germany  
Tel: 49.211.529.30  
Fax: 49.211.529.31.00

**Austria**  
Vienna, Austria  
Tel: 43.1.240.44.160  
Fax: 43.1.240.44.600

**Italy**  
Milan, Italy  
Tel: 39.02.24137.1  
Fax: 39.02.24138.1

**Netherlands**  
Heerlen, The Netherlands  
Tel: 31.45.566.8000  
Fax: 31.45.566.8668

**Spain**  
Madrid, Spain  
Tel: 34.91.625.04.00  
Fax: 34.91.650.74.10

**Scandinavia**  
Järfälla, Sweden  
Tel: 46.8.52.22.00.00  
Fax: 46.8.52.22.00.50

**Asia Pacific**  
Christchurch, New Zealand  
Tel: 64.3.3794.429  
Fax: 64.3.3792.374

**Latin America & Caribbean**  
Weston, Florida USA  
Tel: 954.385.3403  
Fax: 954.306.0176

**Middle East**  
Beirut, Lebanon  
Tel: 961.1.370.670  
Fax: 961.1.364.164

**Hungary**  
Budapest, Hungary  
Tel: 36.1.214.2228  
Fax: 36.1.214.2230

**Poland**  
Warsaw, Poland  
Tel: 48.22.465.69.00  
Fax: 48.22.465.69.17

**Czech Republic**  
Prague, Czech Republic  
Tel: 420.2.2017.2277  
Fax: 420.2.2056.1617

**People's Republic of China**  
Shanghai, China  
Tel: 86.21.50800998  
Fax: 86.21.50800978

**South Africa**  
Bedfordview, South Africa  
Tel: 27.11.677.4800  
Fax: 27.11.616.1104

**Japan**  
Kawasaki, Kanagawa, Japan  
Tel: 81.44.540.6502  
Fax: 81.44.540.6170

**Australia**  
Sydney, Australia  
Tel: 61.2.9879.5999  
Fax: 61.2.9879.5100

**Israel**  
Even-Yehuda, Israel  
Tel: 972.9.891.2223  
Fax: 972.9.891.9330